PORTUGUÊS

Salvavidas importantes

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962-3100-100
Rev. B

This product is protected by U.S. and foreign patents. For a list of patents related to this product please go to <http://amisleep.com/patents>

ENGLISH

Important Safeguards

User should keep these instructions for continued use and care.

Intended Use: The purpose of the AM Aligner is to counteract the forces that represent the most significant side effect of an oral device used for the treatment of obstructive sleep apnea, snoring, and other sleep disordered breathing. The AM Aligner is constructed of a unique thermoplastic material and is to be used every morning to return the teeth to their original position.

Contraindications: Allergic response may occur to the thermoplastic material.

Fitting the AM Aligner

- Remove the AM Aligner from the package. Place the aligner in boiling water for 1 minute. After 1 minute, the Aligner will be soft and flexible.
- Place the softened aligner over upper teeth.

- Bite down firmly on the Aligner into your usual bite position. Try to bite all the way through the AM Aligner. When held to the light, the plastic should be very thin and almost translucent where the tooth indentations are.

- Mold the excess plastic over the upper teeth beginning in the middle and working your way backwards.

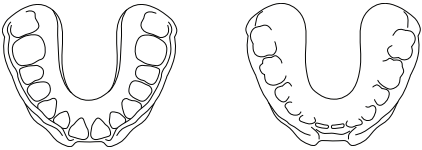
- Using your thumb, create a slight dip in the middle so that the AM Aligner dips down over the bottom front teeth.

- Continue molding to the sides.

- Let set in the mouth for five minutes.
- Carefully loosen the plastic from the back teeth first, then remove the AM Aligner from the mouth. The AM Aligner should have a good shape of the upper arch with deep indentations of the lower arch on the bottom of the tray. Hold the AM Aligner to the light. You should be able to see light through the bite impressions in the tray. If not, follow instructions above to re-fit the AM Aligner and bite harder into the AM Aligner

Daily Use for the AM Aligner

- After removing the treatment appliance in the morning, wait thirty minutes to use the AM Aligner.
- When you are ready to use the AM Aligner, insert the Aligner over your upper teeth.
- Pull your lower jaw back to engage your lower teeth into the indentations on the bottom of the tray.
- Once your lower jaw is in the correct position, squeeze or bite on the tray as if to put your teeth back together.
- Repeat the process of squeezing the jaw together into the AM Aligner until usual bite position is achieved.
- This may take anywhere from 5 to 10 minutes.
- If it takes longer than 10 minutes, take a hot shower or place a warm pack on the joints to warm up the jaw joints and accomplish this process faster.

Contact your prescriber if these methods are not successful.

- To remove the AM Aligner, take the tray off your back teeth first and then remove it from your mouth.

Operating Temperature Range

Do not store or soak at high temperatures (such as a hot car or hot water). This can cause distortion of the appliance.

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